## CE 0123

# METAPIN 3mm

### membrane tacks

## CE 0123

# METAPIN 5mm

### membrane tacks for soft bone


## CE

# METAPIN HOLDER

### tacks positioning device

## INSTRUCTIONS FOR USE

<b>IT</b>	<b>EN</b>
<b>FR</b>	<b>DA</b>
<b>NL</b>	<b>CS</b>
<b>ES</b>	<b>DE</b>
<b>PT</b>	<b>PL</b>
<b>SV</b>	<b>NO</b>
<b>LT</b>	<b>TR</b>
<b>RO</b>	<b>HU</b>
<b>AR</b>	<b>JA</b>
<b>KO</b>	<b>LV</b>
<b>BG</b>	<b>EL</b>
<b>RU</b>	<b>HR</b>
<b>ZH</b>	<b>SL</b>

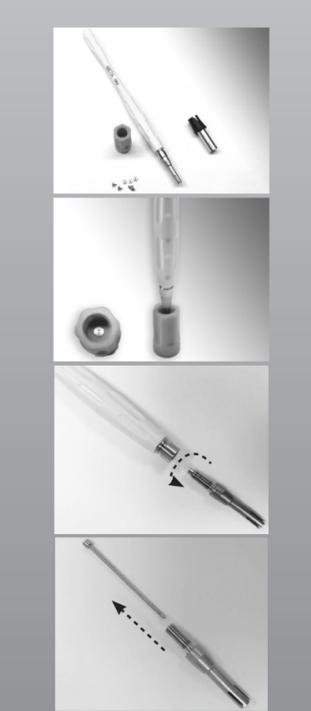REV. 01 - Ottobre 2017

## IT

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## EN

Membrane tacks for bone regeneration in oral surgery.The METAPIN tacks are made of Ti6Al4V titanium alloy to give them high resistance and biocompatibility.

Long-term implantable device for oral surgery applications supplied in sterile packaging. METAPIN is particularly suitable for use in hard mandibular bone without needing to prepare a guide hole. It can be used to stabilise both resorbable and non-resorbable membranes.

- Before use, check that the package is intact.
- Do not use the product if the package is damaged.
- The METAPIN device must be used exclusively by skilled medical staff.
- Always wear sterile gloves when handling the METAPIN device and strictly follow the procedures in order to guarantee sterility.
- The METAPIN device is single-use and absolutely must not be resterilized.
- Warning: if it is reused on another patient, there is a risk of cross-contamination and loss of the performance and functional characteristics of the device.
- Discard after use in special medical waste containers in compliance with the regulations in force.
- META does not shoulder any responsibility for improper use of the product.

Various surgical techniques may be used for device positioning and fixation. The positioning sites and the techniques used are at the discretion of the surgeon. Make sure that you have all the appropriate sterile instruments available before surgery.

- Take METAPIN directly out of the tray with the positioner. The METAPIN tack has a standard gripping system compatible with the standard positioners available on the market. Using the METAPIN HOLDER positioner equipped with a dedicated guide will give the tack more stability during positioning, reducing the risk of it bending/breaking.
- The tack must be removed with an appropriate sterile surgical instrument.

#### Accessories:

The METAPIN HOLDER positioner has a protective cap that must be removed to allow picking up the tack. The cap can also be used to complete tack insertion into the bone by hammering on it if necessary.

The METAPIN HOLDER positioner is supplied non-sterile. It must be disinfected, washed and sterilized before each use. Disinfect it using special germicidal detergents immediately after the surgical procedure in order to prevent possible cross-contamination.

Use suitable detergents for washing. Do not use hydrogen peroxide (H2O2) and chlorinebased detergents. Thoroughly wash the positioner under running water to remove residual tissues using a soft-bristle brush, and check for any damage.

Disinfect and wash the device immediately after each use.

#### Sterilization and storage

The positioner must be sterilized in a steam autoclave at 121°C for 30 minutes. In any case, always use validated sterilization cycles in compliance with the regulations in force. Sterility is guaranteed if the bag is intact; always check it before use.

Only by reading the contents of this document can you prevent errors and ensure proper functioning of our product.

These instructions for use, as well as further information on METAPIN are available on the website: www.metahosp.com

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